

I-POST


New generation
of metallic posts
Steel or Titanium



TITANIUM KIT



STEEL KIT

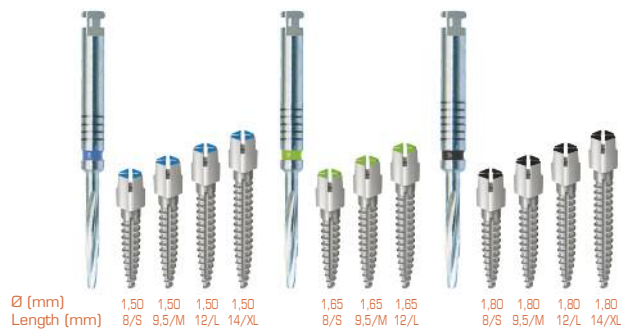

COLOR CODED SYSTEM

- Easy selection of the metallic post and reamer
- Easy arrangement

HELICOIDAL REAMERS

- No vibration
- Perfect gauging
- Efficient dental remain removal

VERTICAL GROOVE
for cement excess
evacuation

TECHNICAL DATA

I-POST TITANIUM


Titanium Grade 5 conform to the standards ISO5832-3 and ASTM F136 (biocompatibility standards for applications of surgical implants such as hip, screws or osteosynthesis spinal implants)

I-POST STAINLESS STEEL

Stainless steel used in surgical instrumentation or for food use, offering a perfect corrosion resistance

- The couple drill / I-Post is designed for a **passive use** to avoid all stress on the root after treatment

The morphology of the I-Post is designed to **respect** the profile of the tooth root (ogival end)

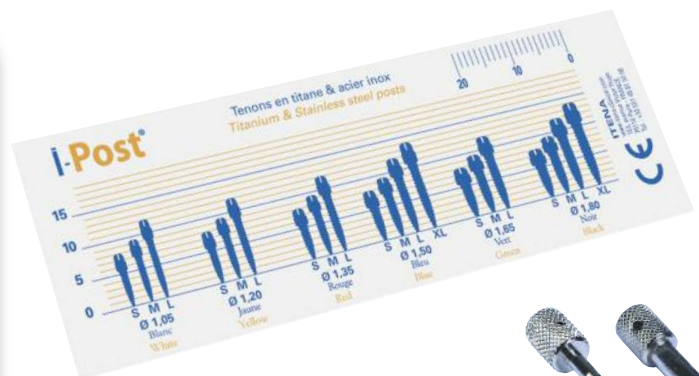


CALIBRATING GAUGE




MANUAL WRENCHES

CLINICAL PROCEDURE

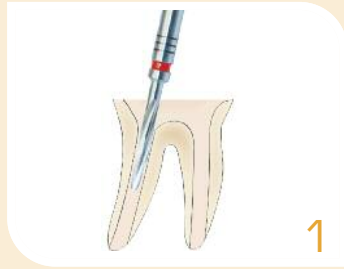

1

Choose the I-Post and select the same color coded reamer

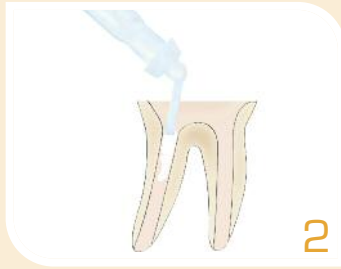

2

Inject cement into the canal



3

Put the I-Post into the canal with the cross or square key. The rotation movement spreads the cement in the canal, eliminates excesses and removes pressures

References

Metallic post refills		Steel per 12	Titanium per 6
Length: 8 mm S	○ ø 1,05	SPIX-8.1A	SPT-8.1A
	● ø 1,20	SPIX-8.2J	SPT-8.2J
	● ø 1,35	SPIX-8.3R	SPT-8.3R
	● ø 1,50	SPIX-8.4B	SPT-8.4B
	● ø 1,65	SPIX-8.5V	SPT-8.5V
	● ø 1,80	SPIX-8.6N	SPT-8.6N

Length: 9,5 mm M	○ ø 1,05	SPIX-9.1A	SPT-9.1A
	● ø 1,20	SPIX-9.2J	SPT-9.2J
	● ø 1,35	SPIX-9.3R	SPT-9.3R
	● ø 1,50	SPIX-9.4B	SPT-9.4B
	● ø 1,65	SPIX-9.5V	SPT-9.5V
	● ø 1,80	SPIX-9.6N	SPT-9.6N

Length: 12 mm L	○ ø 1,05	SPIX-12.1A	SPT-12.1A
	● ø 1,20	SPIX-12.2J	SPT-12.2J
	● ø 1,35	SPIX-12.3R	SPT-12.3R
	● ø 1,50	SPIX-12.4B	SPT-12.4B
	● ø 1,65	SPIX-12.5V	SPT-12.5V
	● ø 1,80	SPIX-12.6N	SPT-12.6N

Length: 14 mm XL	● ø 1,50	SPIX-14.4B	SPT-14.4B
	● ø 1,80	SPIX-14.6N	SPT-14.6N

Reamers	Short	Long
Refill of 3 reamers ○	FSPC-1B	FSPL-1B
Refill of 3 reamers ●	FSPC-2J	FSPL-2J
Refill of 3 reamers ●	FSPC-3R	FSPL-3R
Refill of 3 reamers ●	FSPC-4B	FSPL-4B
Refill of 3 reamers ●	FSPC-5V	FSPL-5V
Refill of 3 reamers ●	FSPC-6N	FSPL-6N
Refill of 6 reamers ○●●●●●	FSPC-assort	FSPL-assort



4

Metallic post kit	Steel	Titanium
Kit of 120 metallic posts with 6 reamers	KSPFIX-126	KSPFT-126



Kit of 120 metallic posts without reamers	KSPIX-120	KSPT-120
---	-----------	----------



Accessories		
1 Square key	CLE-CARREE	
1 Cross key	CLE-CROIX	
20 Stops	BRSP-20	