DENTOCORE

CORE BUILD-UP

DentoCore Core Build-up

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CORE BUILD-UP

DentoCore Core Build-up

Nanofill composite core build-up material. Dual-cure.

INDICATIONS:

DentoCore is a dual-cure, fluoride releasing, resin composite material designed for the fabrication of core build-ups. DentoCore may be used with any Bis-GMA compatible bonding agent.

PROPERTIES:

• Easy to use. • Provides a consistent, homogeneous mix. • Dual curing: material can be light cured, additional chemical cure assures that the material can be quickly bulk filled. • Dense void-free core: cuts like dentin. • Fluoride release: strengthens natural tooth structure and helps prevent secondary caries. • Radiopaque. • Superior compressive strength for long-lasting restorations. • Enhanced mechanical properties of conventional Bis-GMA composite by utilization of Intercalated and Exfoliated nanocomposite technology.

INSTRUCTIONS FOR USE:

1) Prepare the tooth according to clinical requirements. 2) Isolate the tooth from oral fluids. A rubber dam is recommended. 3) Acid etch the entire preparation with a 32%-40% phosphoric acid for 15 seconds or apply DentoBond Quick Adhesive Prime and leave undisturbed for 15 seconds. 4) Rinse off etchant with a water spray for at least 15 seconds. Etched enamel should appear frosted. When using DentoBond Quick Adhesive Prime, remove solvents with a blast of air for at least 10 seconds. 5) Mix equal drops of DentoBond Adhesive or DentoBond Quick Adhesive Bond with the DentoBond Activator and place on the prepared tooth surface. When using another manufacturer's bonding agent, follow instructions for use with dual-cure composites. 6) Allow mixture to sit for 15 seconds and then remove residual solvents with a gentle blast of oil-free air. Repeat application of DentoBond Adhesive / DentoBond Quick Adhesive Bond mixture at least once on the prepared cava tooth surface. It is not necessary to wait 15 seconds between each additional application, however, solvent evaporation is necessary between applications. Light-cure for 20 seconds after final application of bonding agent placed. DentoBond Quick Adhesive Bond contains a unique color chromatic agent which gives the clinician, visual confirmation of polymerization by color change. The chromatic agent is visible (orange) during application and invisible (clear) upon complete polymerization. 7) Place into base of prepared tooth an adequate amount of DentoCore to complete build-up. 8) Do not disturb DentoCore during the gelation stage. Light-cure for at least 40 seconds. Check for complete hardness. Chemical-cure intraoral set-time is approximately three minutes from start of mix application. Set time will be longer outside of mouth.

TECHNICAL DATA:

Working Time:	1:30 minute
Setting Time:	4:30 minute
Compressive strength :	250 MPa
Diametral tensile strength:	40 MPa
Flexural strength:	68 MPa
E-Modulus:	8 GPa
Water sorption:	30 µg/mm ³
Solubility:	2 µg/mm ³

CAUTION!

DentoCore contains polymerisable monomers which may cause allergic contact dermatitis in susceptible persons. Wash thoroughly with soap and water after contact. If skin sensitization occurs, or if known allergy to methacrylate resin exists, discontinue use.

PRECAUTIONS IN USE:

• Do not use in presence of eugenol based materials; complete setting of DentoCore may not take place. • A calcium hydroxide or glass ionomer liner may be used to isolate DentoCore from eugenol sources. • If refrigerated, allow to return to room temperature before use. Refrigeration will increase viscosity, making application more difficult, and will retard setting time.

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