DENTOETCH

ETCHING GEL

F

DentoEtch Etching Gel

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DentoEtch Etching Gel

Etching gel - Phosphoric acid 37%

DentoEtch contains phosphoric acid for etching tooth enamel and dentin when preparing surfaces for the application of composites.

PROPERTIES: • High quality, non-drying, medium viscosity gel. • Stays put, does not run. • Comfortable-to-use syringe. • Tip can be angled to facilitate easy accurate placement of the gel even in distal & lingual composite preparations. • Washes off quickly & easily. • Has excellent water solubility. • Well visible blue color. • Disposable tips save time and are more sanitary.

DIRECTIONS FOR USE: • The effective recommended etching time is : Enamel : 30 seconds. Dentin : as indicated by the manufacturer of the bonding agent. • The DentoEtch should then be rinsed off with copious quantities of water for 30 seconds. • The disposable tip should be removed and discarded after use. • Recap syringe when not in use.

DRYING: • Enamel : dry adequately until the surface has a frosty white appearance. • Dentin : as indicated in the directions for use of the bonding agent.

CAUTION: • DentoEtch contains phosphoric acid. Avoid contact with skin and eyes. If accidental contact occurs, flush with copious amounts of water.

ESSENTIAL INGREDIENTS: • Phosphoric acid 37%. • Water and filler 63%.

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