PERFECT FEEL Flow

FLOWABLE NANO-HYBRID COMPOSITE

Composite fluide nano-hybride photopolymérisable fluoré.

F

INDICATIONS :

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PERFECT FEEL Flow

FLOWABLE NANO-HYBRID COMPOSITE

Flowable and fluorized nano-hybrid light-cured composite.

GB

INDICATIONS:

Filling of minor invasive preparations with carious lesions. • Surface fillings of Class V. Small fillings of Class II. • Liners combined with high viscosity composite materials. • Marginal repairs of current composite fillings or inlays. • Repairs of bis-acrylic crowns and bridges. • Pit and fissure sealant. • Cementing of transparent orthodontic brackets and retainers. • Fixation of teeth after traumatic event or with weakened periodont.

PROPERTIES:

Perfect Feel Flow is a light-cured nano-hybrid composite material with low viscosity, fluoride ion emission and a decreased modulus of elasticity. • Perfect Feel Flow is radiopaque, easy to polish and has the fluorescence of a natural tooth.

CONTRA-INDICATIONS:

Perfect Feel Flow cannot be applied directly to dentine in the proximity of pulp or in an open pulp cavity. • It is not used in fillings with increased occlusal stress. • Perfect Feel Flow is further counter-indicated if a dry field of operation cannot be provided or the prescribed working procedure cannot be adhered to, and in the case of a known hypersensitivity to any of the product's components.

SIDE EFFECTS:

Only in rare cases can reactions from hypersensitivity to a product component be observed with some patients. • No other local or general undesirable side effects were recorded.

INSTRUCTION FOR USE:

Preparation of the field of operation: 1) The tooth surface is first cleaned with a rubber cup and an abrasive paste free from fluorides and oil. It is then rinsed thoroughly in water. 2) The cavity is prepared in an ordinary manner, i.e. with minimum reduction of the dental tissue. The cavity must be sufficiently washed in a stream of water and dried in an air stream free from oils and water. The cavity must be dry and free from debris without over drying the dentine. The use of a rubber dam is recommended for the protection of the working area. It is recommended that dentine in the proximity of the pulp (with deep cavities) be covered with a thin layer of a suitable material, such as a calcium hydroxide cement, to prevent possible damage to the pulp. 3) Apply your adhesive system (Ex: Dentobond Adhesive ITENA) according to the manufacture recommendation. The shade of Perfect Feel Flow is usually selected after cleaning the patient's teeth surface. The shade is determined in daylight with a naturally wet tooth. To provide the natural shade, Perfect Feel flow is available in VITA shades. **Application:** 1) The selected shade of Perfect Feel Flow can be applied directly into the cavity using a syringe, with the application cannulas, or one application quantity is squeezed from a syringe onto a suitable tool. 2) The composite is light cured in layers up to 2 mm in thickness so that the polymerisation contraction is reduced to a minimum. With approximal cavities, the first layer from the buccal or lingual side is cured with a light point. If a metallic matrix is used, the layer to be cured should be no more than 1 mm thick for exposure from the occlusal area only. This technique is repeated until the cavity is filled. Each layer is cured for 20-30 seconds. **Finishing and polishing:** The filling is trimmed with diamond or other fine burs. Polish it with disks or rubber bands. The approximal area is to be polished with fine grained finishing strips. Fine disks and silicon tools can be used to achieve a high gloss on the filling surface.

SHADES:

Syringe of 2 g: A1, A2, A3.5 and translucent.

CAUTION! Direct contact of the product with the skin and eyes must be avoided. Upon accidental contact with the eye, rinse it in water and seek medical assistance. • Perfect Feel Flow may not be used in combination with substances containing eugenol, as eugenol prevents curing. • Mouth washes containing a cation active substance, anti-plaque preparations and chlorhexidine can cause discoloration of the dental tissue, including the filling. • Empty packages and unused material should be disposed of together with ordinary worksite waste.

STORAGE:

Store in a dry and dark place and in a well sealed container. • Store at 5 ° - 23 °C.

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