IPERBOND

ADHESIVE

Iperbond Adhesive

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Iperbond Adhesive

Universal cross generation adhesive

• Choose between an Etch&Rinse, a Self-Etching or a dual cure mode according to the following indications.

INDICATIONS FOR ETCH & RINSE MODE :

This mode is used when bonding implies: Unprepared enamel • Sclerotic dentin • Teeth with a large surface of enamel • Cavity with low retention • Cementation of radicular posts • Bonding of prosthetic pieces (onlays, composites, crowns).

INDICATIONS FOR SELF-ETCHING MODE :

This mode is recommended when the following situations occurs: Enamel and dentin prepared surfaces • Filling cavity with a high retention • Filling of deciduous tooth • Cementing of radicular posts • Bonding without calcium hydroxide paste.

INDICATIONS FOR DUAL CURE MODE :

This mode is indicated for all cases where adhesive is not accessible to the curing light (corono-radicular restorations, bonding of indirect restorations or prosthetic elements).

PROPERTIES :

Iperbond Adhesive is a cross generation dentin/enamel bonding agent which combines etch, primer and adhesive into a single component for ease of application. • Iperbond Adhesive is compatible with all common dental etching materials (Ex. DentoEtch ITENA) and forms an excellent bond with dentin and enamel, any self-cure, dual cure or light-cure composite (Ex. Perfect Feel ITENA), compomer or core build up material (Ex. DentoCore or DentoCore Body ITENA) and with composite resin cements. • Recent studies demonstrate that Iperbond Adhesive is capable of delivering more than 30 MPa shear bond strength to dentin. • Scanning electron micrographs indicate that Iperbond Adhesive application forms the essential hybrid zone on dentin.

INSTRUCTIONS FOR USE :

1) Tooth preparation : Prepare the tooth according to clinical requirements. **2) Etching (Optional) :** a. Etch enamel and/or dentin for 5-10 seconds. When bonding only to enamel, etch enamel for 20-30 seconds. b. Rinse with a spray of oil-free water for 5-10 seconds. c. Blot dry excess moisture from the surface. Verify that the surface has no visible moisture. **3) Application of Iperbond :** a. Dispense Iperbond directly onto a brush or a microbrush by holding the brush tip directly over the nozzle of the tilted bottle. Apply 3 successive coats of Iperbond, leave undisturbed for 15 seconds. b. Remove solvents with a vigorous blast of air for at least 5 seconds or until no visible movement of the resin is noted. c. Cure Iperbond for 20 seconds with a visible light-cure unit. d. Restore with visible light cured or self-cure composite resin of your choice or cement restoration. **4) When light is not accessible :** a. Mix equal drops of Iperbond and Iperbond Activator. b. Apply 3 successive coats of the mixture and leave undisturbed for 15 seconds. c. Remove solvents with a vigorous blast of air for at least 5 seconds or until no visible movement of the resin is noted. d. Cement restoration.

CAUTION !

Iperbond Adhesive contains polymerizable monomers which may cause allergic contact dermatitis in susceptible persons. • Wash thoroughly with soap and water after contact. • If skin sensitization occurs, or if known allergy to methacrylate resin exists, discontinue use.

ESSENTIAL INGREDIENTS :

IPERBOND ADHESIVE : Triethyleneglycol dimethacrylate • Aliphatic polyester urethane acrylate • Adhesive accelerator • Hydroxyethyl methacrylate • Photoinitiators • Acetone • Ethanol • Nano Fillers.

IPERBOND ACTIVATOR : Acetone • Aliphatic polyester urethane acrylate • Oxidizing agent • Nano Fillers.

STORAGE :

• To store between 2 and 8 °C. • Refrigeration is required.

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