

CLINICAL PROCEDURE



1

Select the glass fiber post & matching drill using the calibrating gauge on the drill



2

Prepare the canal with a Gates or a Largo drill



3

Prepare the canal with the calibrated cylindro-conical drill



4

Try the glass fiber post



5

Cut the post with a disk, perpendicular to the post. Keep it in alcohol until bonding



6

Apply the mix on the glass fiber post & surfaces according to the bonding (Iperbond or QuickBond) instructions for use



7

Coat the post with core build-up (DentoCore/ DentoCore Body), then inject the core build-up into the canal. Fill a matrix with DentoCore/DentoCore Body



8

Remove the matrix and make the shape